

### DIRECTIONS FOR USE

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

MadaCide-FD Germicidal Solution is a general purpose cleaner, deodorizer and disinfectant to be used where control of cross-contamination hazards is of prime importance. MadaCide-FD Germicidal Solution provides a novel way to effectively clean and disinfect hard, non-porous environmental surfaces. MadaCide-FD Germicidal Solution aids in the reduction of cross-contamination hazards. Easy to use; no mixing or diluting, except for ultrasonic cleaning. Ready-to-use.

Use MadaCide-FD Germicidal Solution on all hard, non-porous environmental surfaces/articles including: Chrome, Formica® Brand, glass, painted surfaces, plastics (such as polycarbonate, polyvinylchloride, polypropylene and polystyrene), Plexiglas®, glazed porcelain and stainless steel.

Use MadaCide-FD Germicidal Solution to clean, deodorize and disinfect such items as: Ambulance equipment/surfaces, counters/countertops, CPR training mannequins, dental chairs/countertops/instruments/implants/OR surfaces, exercise equipment/benches/mats, hardhats, headphones, laboratory equipment/surfaces, medical instruments/implants, nail care implements, operating room lights/tables, respiratory therapy equipment, respirators/facepieces, tanning beds, telephones and veterinary care surfaces/instruments/implants.



Distributed by:  
**Mada Medical Products, Inc.**  
625 Washington Ave.  
Carlstadt, NJ 07072  
800-526-6370

# MadaCide-FD Germicidal Solution

## Cleaner, Disinfectant & Deodorizer

## Bactericidal, Tuberculocidal, Fungicidal and Virucidal

### ACTIVE INGREDIENTS:

n-Alkyl (60% C14, 30% C16, 5% C12, 5% C18)	
dimethyl benzyl ammonium chloride	0.154%
n-Alkyl (68% C12, 32% C14)	
dimethyl ethylbenzyl ammonium chloride	0.154%

OTHER INGREDIENTS ..... 99.692%

TOTAL ..... 100.000%

### Keep Out of Reach of Children

### CAUTION

See side panel for additional precautionary statements

REORDER NO. 7020

Net Contents: 1 Quart (0.95 Liters)

EPA Reg. No. 1130-15-11703

EPA Est. 1130-IL-1

### DIRECTIONS FOR USE (Continued)

This product has demonstrated effectiveness against Influenza Strain A2/HK virus\* and Avian Influenza A Virus\* and is expected to inactivate all influenza A viruses including Pandemic 2009 H1N1 (formerly called "swine flu").

**GENERAL CLEANING, PRECLEANING AND DEODORIZING:** Prior to disinfection, preclean all surfaces with MadaCide-FD Germicidal Solution, making sure to remove all gross filth and heavy soil. Apply solution to surface and allow surface to remain wet for 30 seconds prior to wiping with a paper towel or clean towel/sponge. **ULTRASONIC CLEANING:** Thoroughly pre-rinse instruments under running water to remove visible gross debris. Using 1 ounce of MadaCide-FD Germicidal Solution per liter of water in ultrasonic unit, immerse instruments into solution and activate ultrasonic unit for 5 minutes (or longer if necessary). Remove and rinse instruments. Follow with appropriate cleaning and disinfection process. Change solution as needed when it becomes diluted or visibly soiled. **TO DISINFECT HARD, NONPOROUS SURFACES/ARTICLES:** To Disinfect *Acinetobacter baumannii*, *Aspergillus niger*, Avian Influenza A Virus, *Candida albicans*, Canine Parvovirus, *Escherichia coli* O157:H7, Hepatitis B Virus (HBV), Hepatitis C Virus (HCV), Herpes Simplex Virus Type 2, HIV-1 (AIDS Virus), Influenza Strain A2/HK Virus, *Mycobacterium bovis* BCG, Pandemic 2009 H1N1 Influenza A Virus\*, Polio 1 Virus, *Pseudomonas aeruginosa*, Rhinovirus, Rotavirus Strain WA, *Salmonella enterica*, *Staphylococcus aureus*, Methicillin-resistant *Staphylococcus aureus* (MRSA), *Trichophyton mentagrophytes* and Vancomycin-resistant *Enterococcus faecalis* (VRE): Using MadaCide-FD Germicidal Solution as is (do not dilute), apply solution to thoroughly wet the pre-cleaned surfaces. Allow to remain wet for 6 minutes prior to wiping or allow surfaces to air dry. For disinfection of Avian Influenza A, use this product only for surfaces that are not conducive to treatment by immersion or excess liquid. **NOTE:** Disinfected food contact surfaces must be rinsed with potable water after cleaning and/or disinfection and before reuse. After disinfection, critical and semi-critical devices must undergo appropriate terminal sterilization/high-level disinfection.

### Special Instructions for Cleaning and Decontamination Against HBV, HCV and HIV-1 (AIDS Virus) for Surfaces or Objects Soiled with Blood or Body Fluids

Kills HBV, HCV and HIV-1 (AIDS Virus) on pre-cleaned environmental surfaces or objects previously soiled with blood or body fluids in 30 seconds at room temperature (20-25°C) in health care settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of HBV, HCV and the human immunodeficiency virus Type 1 (HIV-1) associated with AIDS. **PERSONAL PROTECTION:** When using the germicidal solution, wear disposable latex gloves, protective gown, face mask and/or eye covering, as appropriate, when handling HBV, HCV and HIV-1 infected blood or body fluids. **CLEANING PROCEDURE:** All blood and other body fluids must be thoroughly cleaned from surfaces and objects before application of the germicidal solution. **CONTACT TIME:** Thoroughly wet surface. Allow to remain wet 30 seconds, let air dry. Use a 6-minute contact time for disinfection against all of the organisms claimed. **DISPOSAL OF INFECTIOUS MATERIALS:** Dispose of used solution in accordance with local regulations for infectious waste disposal.

*This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to pre-clean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.*

### STORAGE AND DISPOSAL

Do not contaminate water, food or feed by storage or disposal.

**Storage:** Store in original container in a cool, well-ventilated, locked area inaccessible to children and persons unfamiliar with its use. **PESTICIDE DISPOSAL:** Wastes resulting from the use of this product must be disposed of on site, or at an approved waste disposal facility. **CONTAINER DISPOSAL:** Refillable container; refill this container with only quarternarium ammonium product. Do not reuse this container for any other purpose. Cleaning before refilling is the responsibility of the refiller. Cleaning the container before final disposal is the responsibility of the person disposing of the container. To clean the container before final disposal, empty the remaining contents from this container into application equipment or a mix tank. Fill the container about 10 percent full with water. Agitate vigorously or recirculate water with the pump for 2 minutes. Pour or pump rinse into application equipment or rinse collection system. Repeat this rinsing procedure two more times.

### PRECAUTIONARY STATEMENTS

#### Hazards to Humans and Domestic Animals

**CAUTION:** Harmful if inhaled or absorbed through skin. Causes moderate eye irritation. Avoid breathing vapors and contact with eyes, skin or clothing. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum or using tobacco. Remove contaminated clothing and wash clothing before reuse.

FIRST AID	
<b>IF INHALED</b>	Move person to fresh air. If person is not breathing, call 911 or an ambulance, then give artificial respiration, preferably by mouth-to-mouth, if possible.
<b>IF ON SKIN OR CLOTHING</b>	Take off contaminated clothing. Immediately rinse skin with plenty of water for 15-20 minutes.
<b>IF IN EYES</b>	Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye.
Call a poison control center or doctor for treatment advice. Have the product container or label with you when calling a poison control center or doctor, or going for treatment. You may also contact 1-800-424-9300 for emergency medical treatment information.	

**PHYSICAL OR CHEMICAL HAZARDS:** Do not use or store near heat or open flame.

S5FDQ-80986C-0510